



Introducing the Next
Generation BiLevel by 3B Medical.

Luna G3 BPAP 25A

Integrated heated tubing and modem makes for a streamlined profile and a small footprint. The Luna G3 BPAP 25A is loaded with new features and settings to make bilevel therapy both more comfortable and convenient.



Luna G3 BPAP 25A

SKU: LG3700

GTIN: 00817783021159

 **3B**TM *Medical, Inc.*

Specifications

DEVICE SIZE

Dimensions (L x W x H): 265 mm × 145 mm × 114 mm

Weight: 1.7 kg

Water capacity: To maximum fill line 360 mL

PRODUCT USE, TRANSPORT AND STORAGE

Operation Transport and Storage

Temperature: 5°C to 35°C (41°F to 95°F) -25°C to 70°C (-13°F to 158°F)

Humidity: 15% to 93% Non-condensing 15% to 93% Non-condensing

Atmospheric Pressure: 760 ~ 1060 hPa

MODE OF OPERATION

Continuous

WORK MODE

CPAP, S, AutoS

SD CARD

The SD card can record patient data and fault information

AC POWER CONSUMPTION

100V - 240V ~, 50Hz / 60 Hz, 2 A

DEVICE OFFER TO HEATED TUBING COMMUNICATIONS PORT

24V === 18W

PRESSURE RANGE

4 to 25 cmH₂O (in 0.5 cmH₂O increments)

Under single fault conditions, ≤ 30 cmH₂O for CPAP and AutoCPAP mode,

RAMP

The ramp time ranges from 0 to 60 minutes.

SOUND PRESSURE LEVEL

< 26 dB, when the device is working at the pressure of 10 cmH₂O.

THE FORM AND THE DIMENSIONS OF THE PATIENT CONNECTION PORT

The 22 mm conical air outlet complies with ISO 5356-1.

Replacement Part Numbers

LG31020

Luna G3 Air Filter

LG34510

Luna G3 Water Chamber


LG3B

Luna G3 Carry Bag


LG3HT

Luna G3 Heated Tubing

Ultra Modern Design with a User Friendly Interface

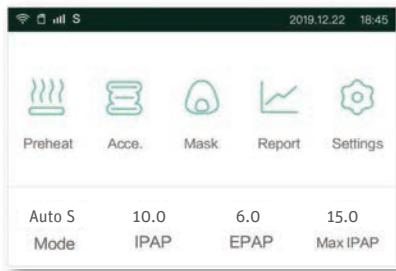

Easy access to frequently used features.

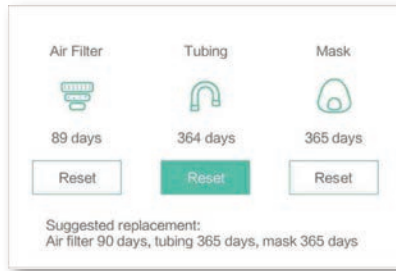



Accessory and supply reminders keep your treatment on track.



Clock, Temperature and Humidity Monitor on stand-by for convenience.

Pre-Heat Feature

The G3 series has a pre-heat feature to warm your humidifier water up for ultimate comfort.

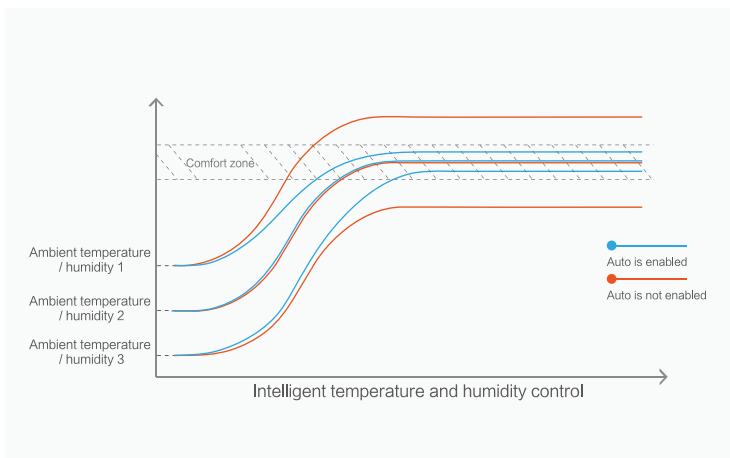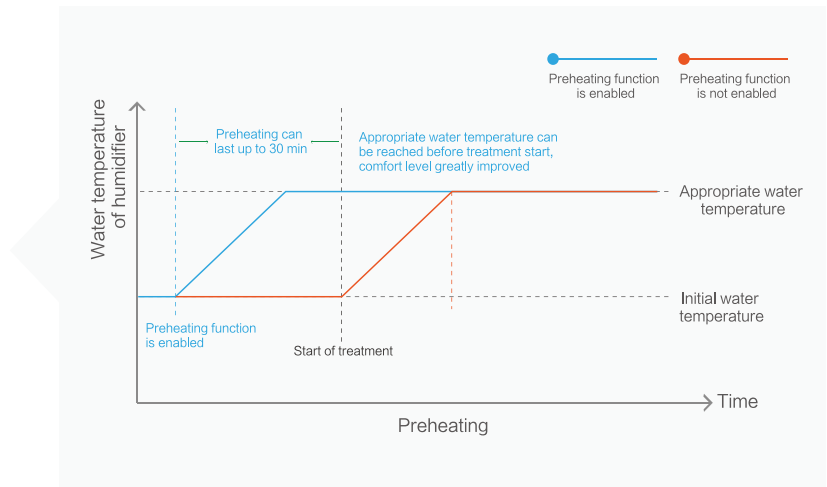


Integrated Heated Tubing

intelligent Auto Humidity control allows G3 maximum moisture while ensuring rain out in the tubing is not an issue.

Auto Ramp

New auto ramp feature senses when the user falls asleep and will adjust ramp time appropriately.

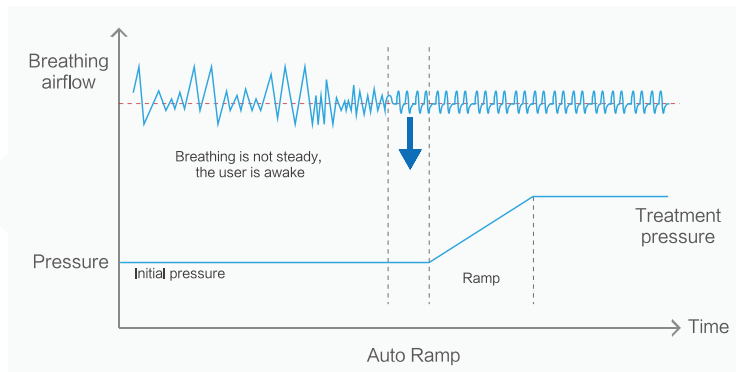

Excellent Sound Performance

Multi-stage noise reduction technology is used to ensure sound performance is under 26 dB.

Data Transfer

